



N-95 MASKS

N-95 masks should be prioritized for high risk patient contact

1 WASH HANDS BEFORE AND AFTER REMOVAL OF MASK

2 PHYSICALLY INSPECT MASK BEFORE USE

3 ENSURE PROPER FIT - NEEDS TO COVER NOSE AND MOUTH

4 PLACE LOWER STRAP BEHIND NECK

5 PLACE TOP STRAP ON BACK OF HEAD

6 AVOID TOUCHING INSIDE OF MASK

7 IF MASK CONTAINS METAL NOSE BAND, BEND METAL TO SHAPE AROUND NOSE WITHOUT PINCHING AIRWAY

8 STORE MASK IN DESIGNATED AREA (BROWN PAPER BAG) OR UNDER CONTROLLED CONDITIONS (UV LIGHT)

9 ENCOURAGE PRACTICAL HYGIENE REUSE OF N-95 MASKS - SHOULD NOT EXCEED 5 USES

10 DISCARD APPROPRIATELY WHEN USE HAS EXPIRED

REFERENCES: PLEASE VISIT
[HTTPS://MANAGEDCAREPHARMACYCA.COM](https://MANAGEDCAREPHARMACYCA.COM)



SURGICAL MASKS

1 WASH HANDS BEFORE AND AFTER REMOVAL OF MASK

2 PHYSICALLY INSPECT MASK BEFORE USE

3 ENSURE PROPER FIT - STARTS AT NOSE BRIDGE AND ENDS DOWN PAST THE MOUTH

4 AVOID TOUCHING INSIDE OF MASK

5 FOLD ENTRANCES SHOULD BE FACING DOWNWARD

6 PLACE LEFT ELASTIC BEHIND LEFT EAR

7 PLACE RIGHT ELASTIC BEHIND RIGHT EAR

8 STORE MASK IN DESIGNATED AREA (BROWN PAPER BAG) OR UNDER CONTROLLED CONDITIONS (UV LIGHT)

9 ENCOURAGE PRACTICAL HYGIENE REUSE MASK

10 DISCARD APPROPRIATELY WHEN USE HAS EXPIRED (24 HOURS)

EMAIL: MHATA@WESTERNU.EDU OR
KEVIN.MANCILLA@WESTERNU.EDU